INSTRUCTION MANUAL FOR
COMPRESSOR NEBULIZER

GC806

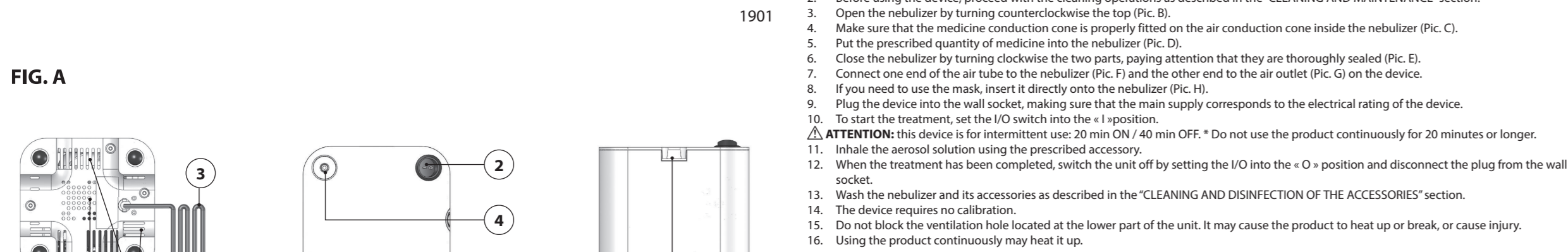
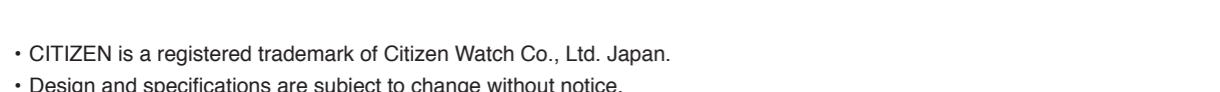
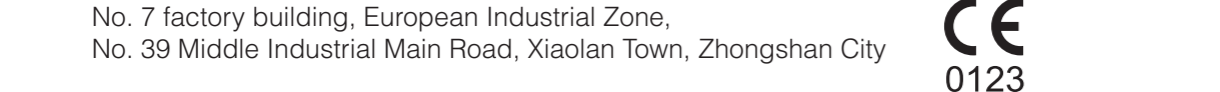
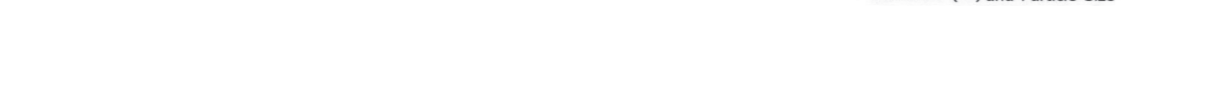
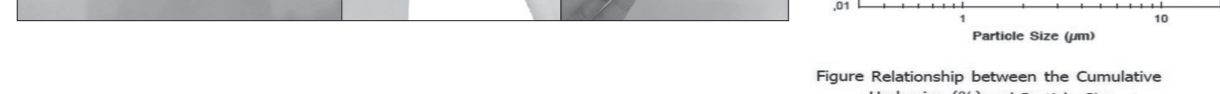
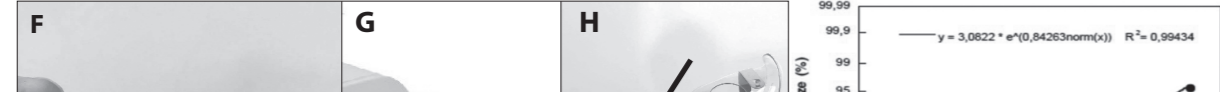
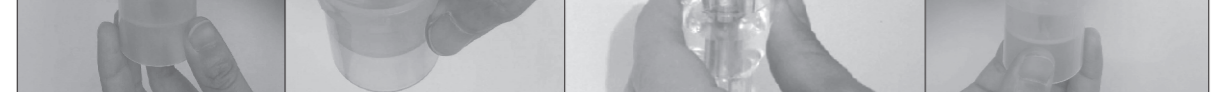
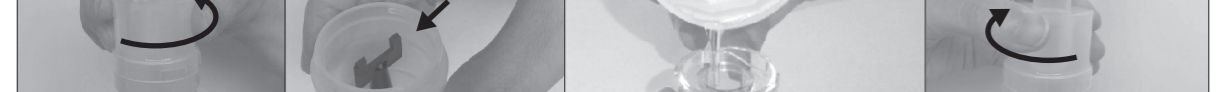
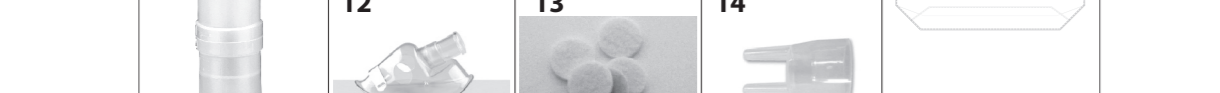
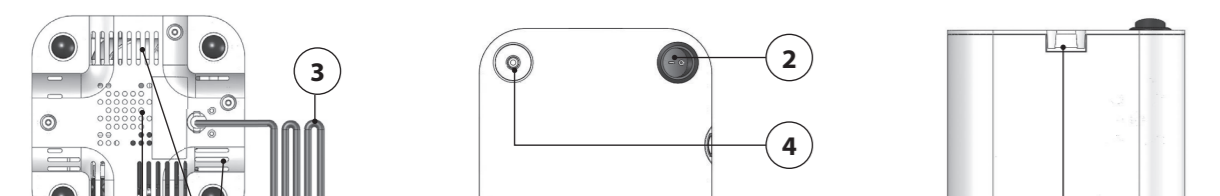
FIG. A

English	GC806 COMPONENTS - (PIC. A)	Spanish	GC806 COMPONENTES - (FIG. A)
1. Compressor housing	6. Nebulizer holder	11. Adult mask	11. Mascarilla para adultos
2. ON/OFF switch	7. Air-vent openings	12. Pediatric mask	12. Mascarilla pediátrica
3. Power cord	8. Nebulizer	13. Spare air filters	13. Filtros de aire de repuesto
4. Air outlet	9. Mouthpiece	14. Nospiece	14. Pieza nasal
5. Filter	10. Air tube	15. Accessories case	15. Contenedor porta-accesorios

IMPORTANT: READ CAREFULLY THE INSTRUCTIONS FOR USE BEFORE OPERATION.
GENERAL INFORMATION
 GC806 is an aerosolizing system suitable for domestic use.
 Use the device only as described in this manual and therefore as an aerosol therapy system, following the indications of your doctor. Any use different from the intended one is to be considered improper and hence dangerous; the manufacturer cannot be held liable for any damage caused by improper, incorrect and/or unreasonable use, or if the equipment is connected to electrical installations which do not comply with current safety regulations.

- Keep this manual for future reference.
- Do not operate the unit in presence of any anaesthetic mixture inflammable with oxygen or nitrogen protoxide.
- The correct functioning of the equipment can be affected by electromagnetic interferences which exceed the limits indicated by the European standards in force. In case this device interferes with other electrical devices, move it and plug it to a different power socket.
- In case of failure and/or malfunction, read the "POSSIBLE PROBLEMS AND HOW TO SOLVE THEM" section. Do not tamper or open the compressor housing.
- Use the dedicated spare parts only.
- Comply with the safety regulations concerning the electrical devices and in particular:
 - use only unicast accessories and components;
 - never submerge the unit in water;
 - never wet the device. It is not protected against sprinkles;
 - never touch the unit with wet or moist hands;
 - do not leave the unit exposed to the weather elements;
 - place the unit on a stable and non-conductive surface;
 - if the product is dropped by your foot or hand accidentally catching the power cord, it may cause damage or injury. Keep the product in a stable place and be careful when moving it;
 - the use of this device by children and disabled requires always the close supervision by an adult with full mental faculties;
 - do not pull the power cord or the device itself to unplug it from the power socket;
 - the power plug is the separation element from the grid power; keep the plug accessible when the device is in use;
 - do not block the air vent for the compressor housing.
- Before plugging in the device, make sure that the electrical rating, shown on the rating plate on the bottom of the unit, corresponds to the mains rating.
- In case the power plug provided with the device does not fit your wall socket, address to qualified personnel for the replacement of the plug with a suitable one. In general, the use of adapters, simple or multiple, and/or extension cable is not recommended. If their use is indispensable, it is necessary to use that complying with safety regulations, paying however attention they do not exceed the maximum power limits, indicated on adapters and extension cables.
- Do not leave the unit plugged in when not in use; unplug the device from the wall socket when it is not operated.
- The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or things, for which the manufacturer cannot be held responsible.
- The power cord of this device cannot be replaced by the user. In case of a power cord damage, address to a technical service centre authorized by the manufacturer for its replacement.
- The power supply cord should always be fully unwound in order to prevent dangerous overheating.
- Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.
- Some parts of the unit are so small that they will be swallowed by children; keep the equipment out from children's reach.
- If you decide not to use the device any longer, it is recommended to dispose of it according to the current regulations.
- Make sure to:
 - use this device only with medicines prescribed by your doctor;
 - make the treatment using only the accessory recommended by your doctor depending on the pathology;
 - use the nasal fork accessory only if expressly indicated by your doctor and paying attention to NEVER introduce the bifurcations in the nose, but only bringing them as close as possible.
- Check in the medicine package leaflet for possible contraindications for use with common aerosol therapy systems.
- To avoid strangulation and entanglement, keep cable and air tubes out of reach of young children.
- Do not position the equipment so that it is difficult to operate the disconnection device.
- For greater hygienic safety, we recommend you to avoid using the same accessories for more than one person.
- Insert the air tube firmly into the air outlet. The air tube may come off when using the product.
- If you use it continuously for a long period of time, the compressor housing may heat up.

- INSTRUCTIONS FOR USE**
- The device must be checked before each use, in order to detect possible functioning anomalies and/or damages due to transport and/or storage.
 - During inhalation, sit upright and relaxed at a table and not in an armchair, in order to avoid compressing your respiratory airways and impairing the treatment effectiveness.
 - The accessories must be used only with a single patient; it is not recommended to use them with several patients.
- After unpacking the device, check it for visible damages or defects; pay particular attention to cracks in the plastic housing, which may expose electrical components. Check for accessories integrity.
 - Before using the device, proceed with the cleaning operations as described in the "CLEANING AND DISINFECTION OF ACCESSORIES" section.
 - Open the nebulizer by turning counterclockwise the top (Pic. B).
 - Make sure that the medicine conduction cone is properly fitted on the air conduction cone inside the nebulizer (Pic. C).
 - Put the prescribed quantity of medicine into the nebulizer (Pic. D).
 - Close the nebulizer by turning clockwise the two parts, paying attention that they are thoroughly sealed (Pic. E).
 - Connect one end of the air tube to the nebulizer (Pic. F) and the other end to the air outlet (Pic. G) on the device.
 - If you need to use the mask, insert it directly on the nebulizer (Pic. H).
 - Plug the device into the wall socket, making sure that the main supply corresponds to the electrical rating of the device.
 - To start the treatment, set the I/O switch into the "I" position.
 - ATTENTION:** this device is for intermittent use: 20 min ON / 40 min OFF. * Do not use the product continuously for 20 minutes or longer.
 - Inhalate the aerosol solution using the prescribed accessory.
 - When the treatment has been completed, switch the unit off by setting the I/O into the "O" position and disconnect the plug from the wall socket.
 - Wash the nebulizer and its accessories as described in the "CLEANING AND DISINFECTION OF THE ACCESSORIES" section.
 - The device requires no calibration.
 - Do not block the ventilation hole located at the lower part of the unit. It may cause the product to heat up or break, or cause injury.
 - Using the product continuously may heat it up.

CLEANING AND MAINTENANCE
 The cleaning of the device must be carried out by using a soft and dry cloth and non-abrasive cleaners.
ATTENTION:
 During cleaning operations, make sure the internal parts of the unit are not in contact with liquids and that the power plug is disconnected.

CLEANING AND DISINFECTION OF THE ACCESSORIES
 Follow carefully the cleaning and disinfection instructions of the accessories as they are very important for the device performances and the therapy success.

- First use and after each treatment:**
 Disassemble the nebulizer by turning counterclockwise the top and remove the medicine conduction cone. Wash the components of the disassembled nebulizer with the mouthpiece, the mask and the nospiece by using tap water in boiling water for 5 minutes. Reassemble the nebulizer components and connect it to the air-outlet (Pic. G), switch the device on and let it work for 10 - 15 minutes.
- Do not boil nor autoclave the air tube and masks.
- Sterilization:**
 Cold disinfecting liquids may be used instead of boiling water, following the manufacturer's instructions.

REPLACEMENT OF THE NEBULIZER
 The nebulizer must be replaced after a long period of inactivity. In case it shows deformations or breakings, or when the nebulizer is obstructed by dry medicine, dust, etc. It is recommended to replace the nebulizer after a period between 6 months and 1 year depending on the use. Use original nebulizers only.

REPLACEMENT OF THE AIR FILTER
 In normal usage conditions, the air filter must be replaced approximately after 500 working hours or after each year. It is recommended to periodically check the air filter (10 - 12 treatments) and, if the filter shows a grey or brown colour or is wet, replace it. Extract the filter (Pic. A-5) and replace it with a new one. Do not try to clean the filter for reusing it. The air filter shall not be serviced or maintained in use with a patient.

REPLACEMENT OF THE ACCESSORY KIT
 No. 8 to 14 in the figure are consumables. They are available to be purchased separately (model number: AT-GC202). When purchasing any of the above items, contact your local distributor.

Do not use the device without filter.
POSSIBLE PROBLEMS AND HOW TO SOLVE THEM.

- The device does not switch on.
- The device does not nebulize or nebulizes weakly

SOLUTIONS:
 1. Make sure the power plug is firmly fitted to the wall socket.
 2. Make sure that the device has been operating without any indication in this manual (20 min ON / 40 min OFF).
 3. Make sure that the ends of the air tube are tightly fitted to the main unit and nebulizer;
 4. Verify that the nebulizer is not empty or is filled with the proper quantity of medicine (max 8 ml).
 5. Verify that the nebulizer nozzle is not obstructed.

The device not start working properly again, address to local distributor.

MAINTENANCE AND REPAIRS
 In case of failure, address to local distributor. Do not open the device in any case. The unit has no user-serviceable parts within and does not need internal maintenance or lubrication.

TECHNICAL SPECIFICATIONS
 Model: GC806
 Power supply: Please refer to the rating label.
 Minimum filling volume: 2 ml
 Maximum filling volume: 8 ml
 Nebulization rate: 0.35 ml/min (average)
 Maximum pressure: 2.2 bar
 Weight: 1.5 kg
 Size: 130 x 130 x 117 mm
 Noise level: 52 dBA (average)
 Expected service life: 1000 hours

Class II device as regards protection against electric shocks.
 Nebulizer, mouthpiece and masks are type BF applied parts.
 Device not protected against sprinkles.
 Device not suitable for use in presence of anaesthetic mixtures inflammable with air, oxygen or nitrogen protoxide.

Device for intermittent use (20 min ON / 40 min OFF)
 Device not suitable for use in anaesthesia or lung ventilation systems.
 NOTE: The technical specifications may change without prior notice.

AEROSOL CHARACTERISTICS ACCORDING TO 13544-1 ANEXO CC
 Flow: 5.31 l/min
 Aerosol output: 0.11 ml
 Aerosol output rate: 0.07 ml/min
 Particle size (MMAD): 3,07 µm

APPLIED STANDARDS
 • Electric Safety Standards EN 60601-1
 • Electromagnetic Compatibility according to EN 60601-1-2
 • Class IIa Medical Device according to the 93/42/EEC Directive
 • Directive on Medical Devices

ENVIRONMENT CONDITIONS
Storage
 Temperature: MIN -20 °C - MAX +60 °C
 Humidity: MIN 10% RH - MAX 95% RH
 Atmospheric pressure: 700 hPa - 1060 hPa

OPERATION
 Temperature: MIN +10 °C - MAX +40 °C
 Humidity: MIN 10% RH - MAX 95% RH
 Atmospheric pressure: 700 hPa - 1060 hPa

DEVICES SYMBOLS
 I ON
 O OFF
 ~ ALTERNATING CURRENT
 II CLASS II EQUIPMENT
 [A] TYPE BF APPLIED PART
 CE0123 COMPLYING WITH THE 93/42/EEC DIRECTIVE

MANUFACTURER
 SERIAL NUMBER
 IP21 Classification for water ingress and particulate matter.
 ATTENTION
 RELATIVE HUMIDITY
 TEMPERATURE LIMITATION
 REFER TO INSTRUCTIONS

Correct Disposal of This Product (Waste Electrical and Electronic Equipment) (Applicable in the European Union countries and countries with separate collection systems) This symbol on the product or its packaging indicates that the product complies with the electrical equipment and should not be disposed of with household waste. The user is responsible for assigning the end of life, to the appropriate collection, subject to the penalties provided for in current legislation on waste. For further information regarding the collection systems available contact your local waste disposal service.

CITIZEN SYSTEMS JAPAN CO., LTD.
 6-1-12, Tanshi-cho, Mishu-Tokyo-shi,
 Tokyo 185-8511, Japan
 E-mail: sales-oe@systems.citizen.co.jp
 http://www.citizen-systems.co.jp/

CITIZEN is a registered trademark of Citizen Watch Co., Ltd. Japan.
 Design and specifications are subject to change without notice.

English	GC806 COMPONENTS - (PIC. A)	Spanish	GC806 COMPONENTES - (FIG. A)
1. Compressor housing	6. Nebulizer holder	11. Adult mask	11. Mascarilla para adultos
2. ON/OFF switch	7. Air-vent openings	12. Pediatric mask	12. Mascarilla pediátrica
3. Power cord	8. Nebulizer	13. Spare air filters	13. Filtros de aire de repuesto
4. Air outlet	9. Mouthpiece	14. Nospiece	14. Pieza nasal
5. Filter	10. Air tube	15. Accessories case	15. Contenedor porta-accesorios

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